

CONTRAINDICATIONS for the
Noblerex K-1 *Platinum* Whole Body Vibration Machine

- Acute Hernia, Discopathy, or Spondylosis
- Acute Thrombosis
- Artificial Joints (recent)
- Epilepsy
- Head Injuries, Known Neurological Conditions
- Pacemaker and Implantable Cardioverter Defibrillator
- Pregnancy
- Pulmonary Embolism
- Recent Wounds from an Operation or Surgery
- Recently Placed IUD's, Metal Pins, or Plates
- Retinal Detachment, Known Retinal Conditions
- Serious Cardiovascular Disease
- Severe Migraines
- Tumors (cancerous)
- Type I Diabetes

Warning:

Read and follow all instructions and warnings and obtain proper instruction prior to using this equipment. If you have any of the conditions listed above please consult a physician or physical therapist before using the K-1 *Platinum*. This device may not be suitable for all users. Strong vibration affecting the neck and head can occur when using positions other than standing, consulting a physician for approval is recommended. Misuse of this machine may result in serious injury. Keep children away from this equipment. If you feel any unusual pain or discomfort, discontinue use and consult your physician. **User weight must not exceed 330lbs.**

Caution:

Place Equipment on a level surface to avoid damage, breakdown, and/or excessive noise. Do not expose the K-1 *Platinum* unit to rain or moisture. When the K-1 *Platinum* is not used for an extended period of time, remove the power plug from outlet.